



## OMI PADS pulsed electromagnetic mattress



Full body PEMF mat  
for the overall well-being of the body

### Description

The modern, easily foldable mattress comes with an operation unit. The device works in the frequency range from 1 to 99 Hz and it has three different programmes. Mattress size: 165 x 65 cm. Values of the magnetic field intensity measured on the surface of the device: OMI Pads (99 Hz): 220 microtesla; OMI Pads (8 Hz): 210 microtesla. Waveform: sinus. Powered by DC power supply.

### Recommended for

The mattress is recommended for people who live stressful life, feel less energetic and wish to improve their overall well-being.

### Claims

The pulsing magnetic field promotes the body's own regeneration processes and helps

- ✓ to maximize mental clarity, focus and productivity
- ✓ to increase energy and vitality
- ✓ to aid relaxation
- ✓ to sleep better and have better general mood

### Advantages

Ergonomic and foldaway design, for both professional and home use. Completely safe, not painful to use, and there are no known side-effects.

### Instructions for use

The use of the product is very easy, no professional skills are needed. Simply choose the programme, set the length of the usage, and then, just lie down on the mattress. It is recommended to use the mattress 2 to 3 times every day for 20 to 30 minutes per occasion. Due to the low-intensity of the pulsed electromagnetic fields, you can't even feel it during the sessions.

Three different programmes are available: P1 (default programme, the frequency gradually increases from 3 Hz to 38 Hz and then drops back), P2 (default programme, the frequency continually and randomly changes between 1–99 Hz), P3 (the frequency is adjustable between 1 and 99 Hz).

### How it works

Pulsed electromagnetic fields produced by the device induce electrical current inside the body that helps to restore the blocked ion flow and stimulates the body's own regeneration processes.

As a result of using the mattress the body attains a higher energy level, the fatigue is alleviated, helping to achieve improved overall well-being of the whole body.

### Packaging contains

Mattress, operating unit, wall adapter, user manual.

## Documentation and others

Declaration of Conformity (LVD, EMC)

## Wholesale Inquiries

All level of wholesale inquiries are welcome. To discuss further details, please [contact us](#).

**Payment Terms:** on delivery

**Pick-up Point:** Budapest

**Minimum Order Quantity:** [ask for quote](#)

**Country of Origin:** European Community

**Delivery Term, Currency and Price:** [ask for quote](#)

## Package Details

carton with plastic handle



**Product Dimensions (cm):** 81 x 43 x 9 cm

**Product Net Weight (kg):** 2.3 kg

**Product Gross Weight (kg):** 3.3 kg

**Master Carton Dimensions (cm):** 83 x 45 x 47 cm

**Master Carton Gross Weight (kg):** 18.5 kg

**Master Carton Quantity (pc):** 5 pcs

*The product is not a medical device; it is not intended to diagnose, treat, cure or prevent any disease.*

product category: wellness product

**website:** [www.oxfordmedicals.com/pemf-devices.html](http://www.oxfordmedicals.com/pemf-devices.html)

