



## OMI RING pulsed electromagnetic device



### PEMF Therapy Ring for the overall well-being of the body

#### Description

Annular shaped electromagnetic device with such ergonomic design that allows to pull it over the head down to the neck, or slip a limb through it. It comes with an operation unit. The device works in the frequency range from 1 to 99 Hz and has three different programs. The maximum value of the magnetic field strength (measured on the surface of the device) is 210 microtesla. Waveform: sinus. Diameter: 28 cm. Powered by DC power supply.

#### Recommended for

The OMI RING is recommended for people who live stressful life, feel less energetic and wish to improve their overall well-being.

#### Claims

The pulsing magnetic field promotes the body's own regeneration processes and helps

- ✓ to improve your physical and mental well-being,
- ✓ to increase energy and vitality,
- ✓ to have a better general mood.

#### Advantages

For both professional and home use. Completely safe, not painful to use, and there are no known side-effects.

#### Instructions for use

The use of the product is very easy, no professional skills are needed. Simply choose the programme, set the length of the usage, and then, just feel the beneficial effect of the pulsed electromagnetic fields. It is recommended to use the device 2 to 3 times every day for 20 to 30 minutes per occasion. Due to the low-intensity of the pulsed electromagnetic fields, you can't even feel it during the sessions.

Three different programmes are available: P1 (default programme, the frequency gradually increases from 3 Hz to 38 Hz and then drops back), P2 (default programme, the frequency continually and randomly changes between 1–99 Hz), P3 (the frequency is adjustable between 1 and 99 Hz).

#### How it works

Pulsed electromagnetic fields produced by the devices induce electrical current inside the body that helps to restore the blocked ion flow and stimulates the body's own regeneration processes.

As a result of using the device the body attains a higher energy level, the fatigue is alleviated, helping to achieve improved overall well-being of the whole body.

#### Packaging contains

Pulsed electromagnetic ring, operation unit, wall adapter, user manual.

## Documentation and others

Declaration of Conformity (LVD, EMC)

## Wholesale Inquires

All level of wholesale inquiries are welcome. To discuss further details, please [contact us](#).

**Payment Terms:** on delivery

**Pick-up Point:** Budapest

**Minimum Order Quantity:** [ask for quote](#)

**Country of Origin:** European Community

**Delivery Term, Currency and Price:** [ask for quote](#)

## Package Details



**Dimensions (cm):** 29 x 29 x 10 cm

**Net Weight (kg):** 0.96 kg

**Gross Weight (kg):** 1.26 kg

**Master Carton Dimensions (cm):** 64 x 60 x 46 cm

**Master Carton Quantity (pc):** 16 pcs

**Master Carton Gross weight (kg):** 22.2 kg

*The product is not a medical device; it is not intended to diagnose, treat, cure or prevent any disease.*

product category: wellness product

website: [www.oxfordmedicals.com/pemf-devices.html](http://www.oxfordmedicals.com/pemf-devices.html)



opr-pis-gb-190408 • v4.2

Manufactured by  
**Oxford Medical Instruments Limited**  
2<sup>nd</sup> Floor Curzon House, 24 High Street, Banstead, Surrey SM7 2LJ, England

[www.oxfordmedicals.com](http://www.oxfordmedicals.com)